



Osstell IDx | Forgot PIN Code



- 1 Start your IDx. When prompted to enter your PIN code, press the symbol in the upper left corner.

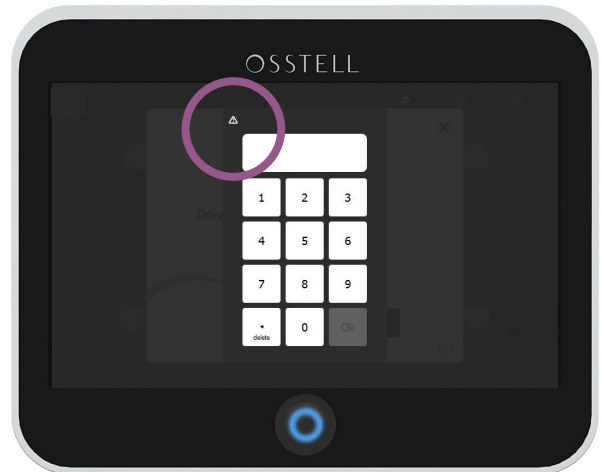

- 2 Enter your PUK code and press enter.

NOTE! If you don't have the PUK code, please contact; support@osstell.com

- 3  Click the settings symbol found in the lower left corner and then select "More".

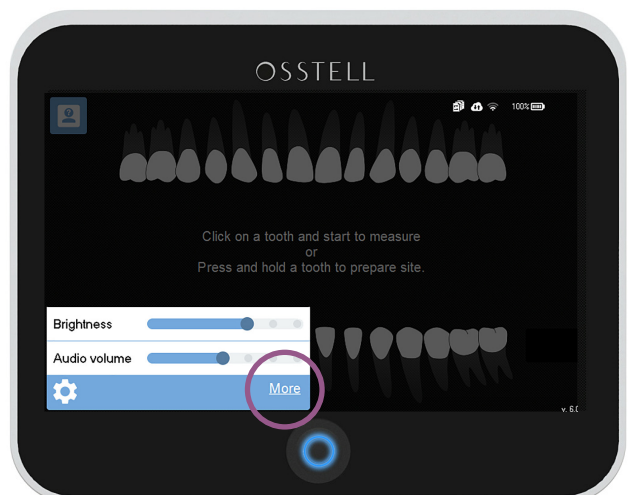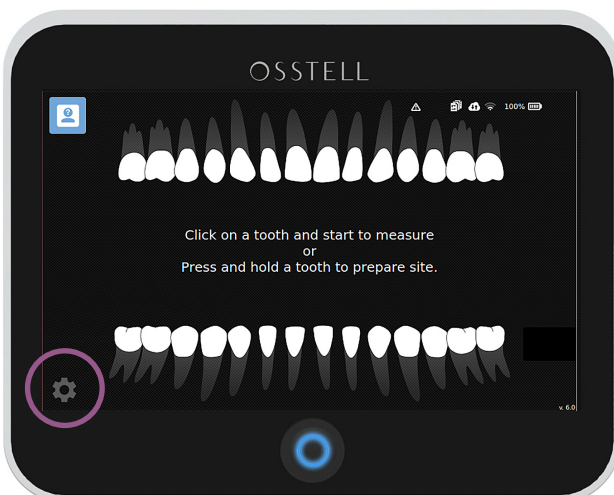

- 4 Under Settings meny, Security, press the **** field next to the PIN. Then select a new PIN code and press Ok.



You have now successfully changed your PIN code. Restart your IDx and enter your new PIN when prompted to.

Let's start measuring again!