


# Osstell IDx | How to connect to the Internet

There are a few different ways you can connect your IDx to the Internet to access, share and backup data among other things.

## Connecting to WiFi


- 1 Tap the settings symbol in the bottom left corner and then select "More". 




- 2 Select the tab "net service", make sure network is enabled and then click "WiFi name" to select network.




- 3 Select WiFi network from the list, or click "manual" to enter SSID

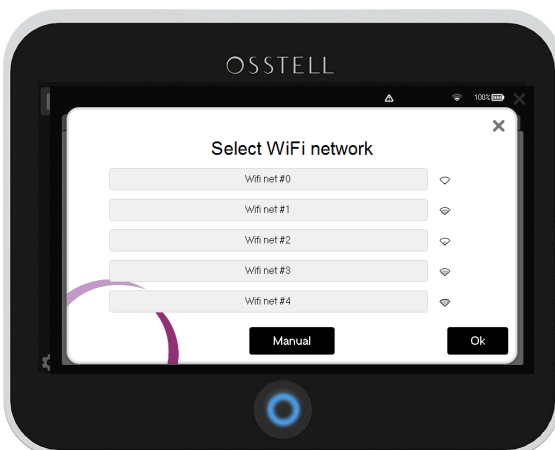

- 4 Enter WiFi password and click "Ok".

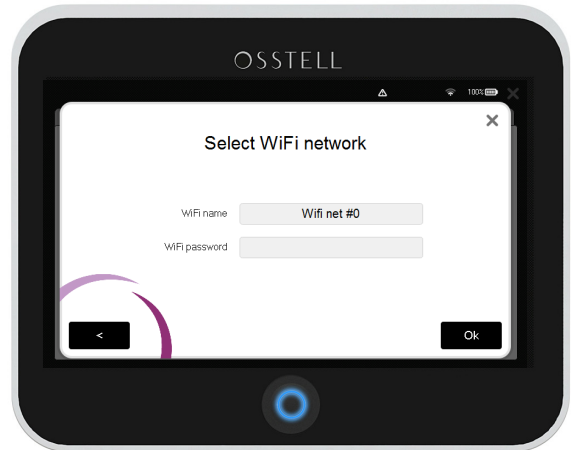

- 5 Once the connection has been established, the WiFi symbol and the cloud will be highlighted.

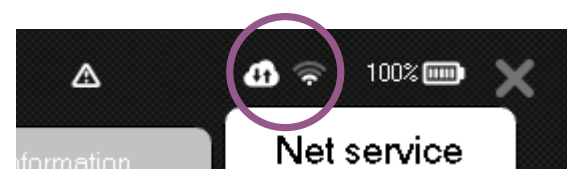

## Connecting to WiFi via your iPhone or Android phone


1



### Set up Personal Hotspot

1. Tap Settings > Mobile Data.
2. Tap Personal Hotspot and turn it on.

After you set up Personal Hotspot, you can go to Settings > Personal Hotspot to turn it on or off and change your password.

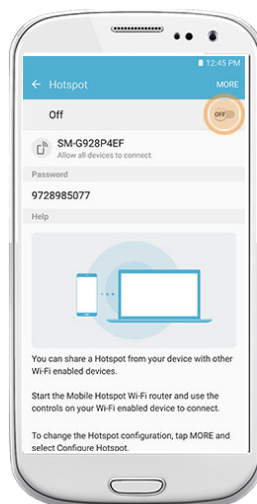

### Set up Mobile Hotspot

1. Tap Settings > Wireless & Networks > More
2. Select Tethering & Portable Hotspot and then check the Portable Wi-Fi Hotspot or Mobile Hotspot item.
3. Choose the Set Up Wi-Fi Hotspot item to give the hotspot a name and assign a password.

2

You have now created a personal WiFi network. Please follow the steps outlined on the previous page on how to connect your IDx to that WiFi.

## Connecting to the Internet via a network cable

1

Plug in the network cable / ethernet cable to the back of your IDx. Once the connection has been established, you will see a highlighted wire symbol in the top right corner. The cloud symbol should also be highlighted.

