


Osstell IDx | Sharing data

If you have more than one Osstell IDx, you are able to share patients between instruments. You can, for example, perform the first measurement with one IDx, and then do the subsequent follow-up measurements on a different IDx.

This is how you get started:



1

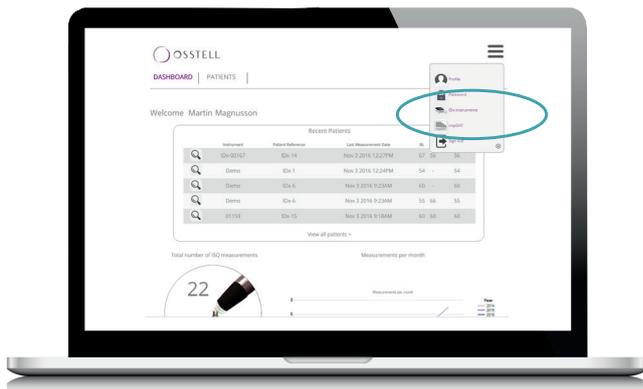



Log in to your OsstellConnect.com account and select "IDx instruments" in the menu.

Please note that your IDx's need to be connected to the Internet to ensure proper sync between the instruments.

2

Click "Activate Sharing" to connect your instruments. You can set your own name for the different instruments for easier identification.

To stop sharing data, simply click "Decactivate Sharing"

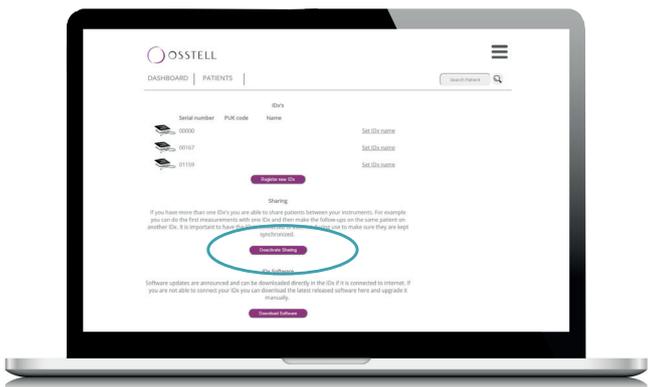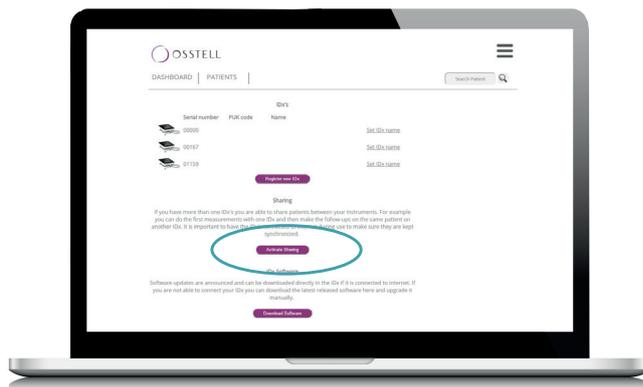

The example below shows data sharing between two Osstell IDx (S/N's 00000 and 00010).

