


Osstell Beacon | How to register your instrument


1 Create your Osstell Connect account

Go to www.osstellconnect.com/register
Follow the steps.

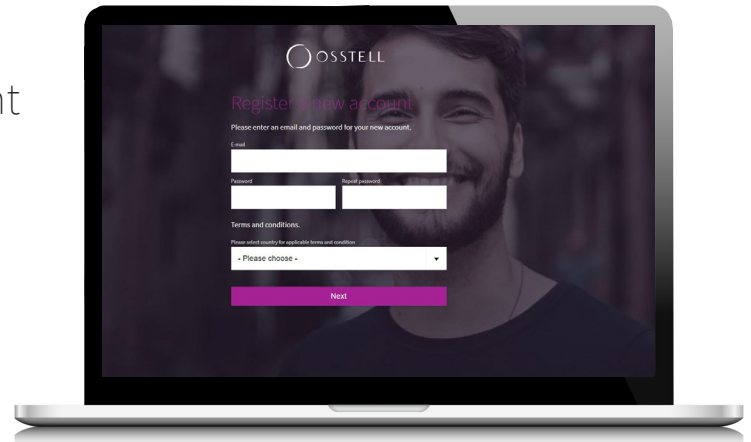


2 Download the gateway

Go to "Devices"
Click "Download gateway".
Install by following the steps.

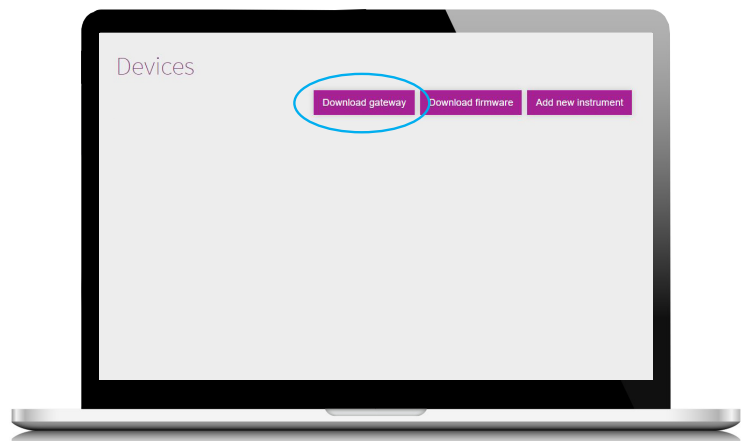


3 Plug in the Osstell Key


Set the Osstell Key (A)
to a USB port on your computer.


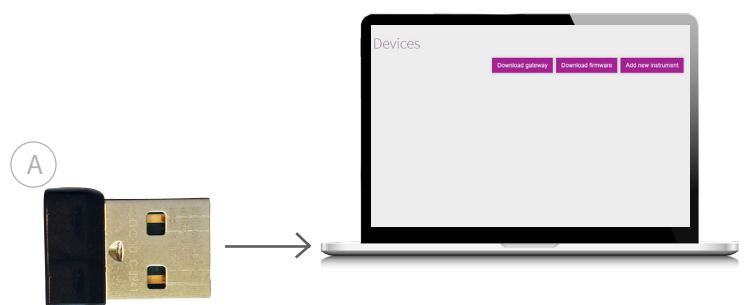


4 Register your instrument

Click "Add new instrument".
A pop-up window will show,
click on the Beacon picture.

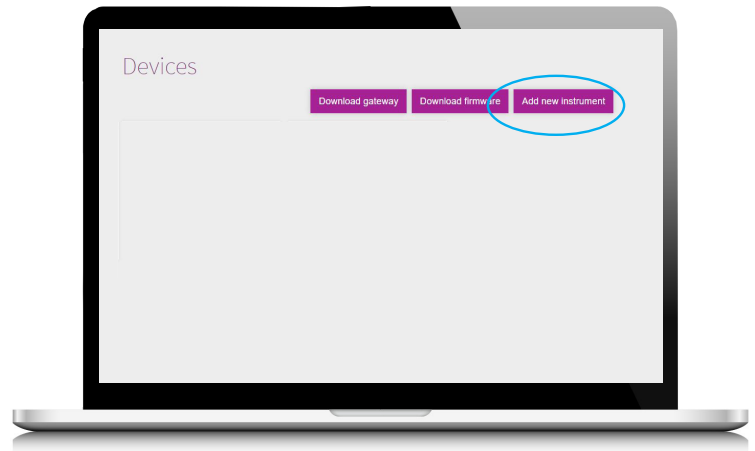

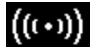
Fill in the serial number which you can find on the back of the instrument (A).
Fill in the the registration number. To find it, take your instrument in hand, the registration number is shown on the lower display (B). Please note that this step can take a few seconds, as your instrument is establishing connexion. When connected, a symbol  will show.

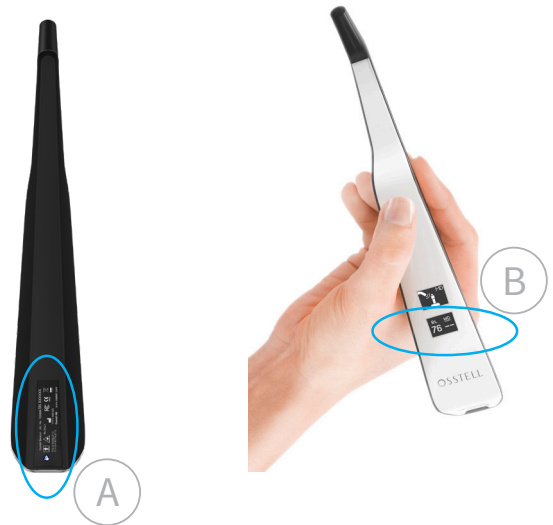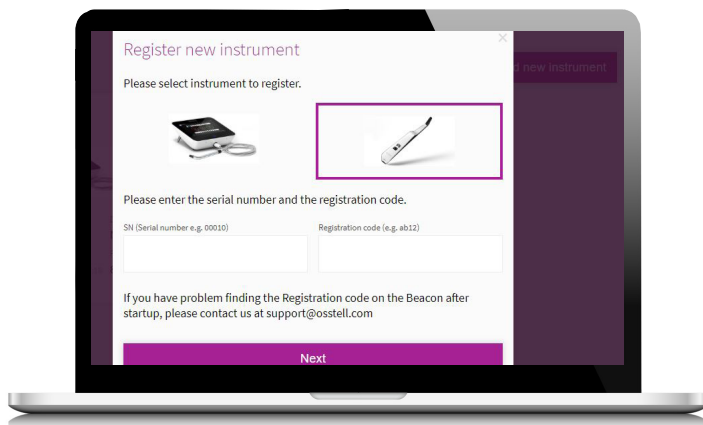

Get free SmartPegs



Click "Get free SmartPegs".
Select your SmartPeg Type.
Fill in your contact details.
Click done.
We will send you SmartPegs.

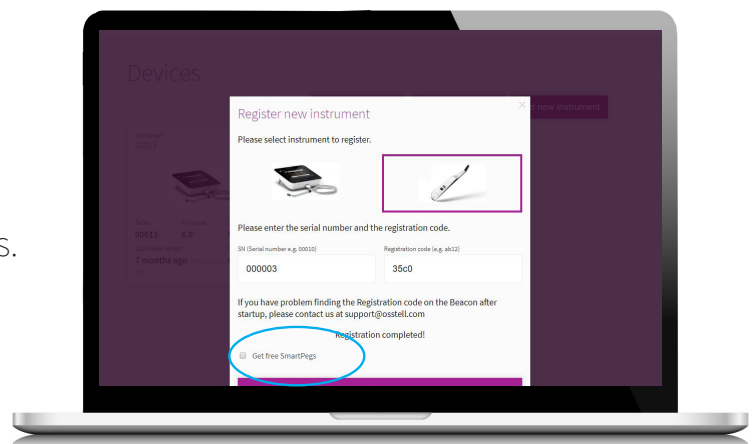

Congratulations!